Anti-Nausea Wristbands

- Drug-free alternative for motion sickness and nausea
- Ideal for air, boat or car travel and roller coasters
- Works by exerting a gentle pressure on the NEI-KUAN acupressure point on each wrist
- Recommended for adults or children over the age of 12 (for more options, please see PS452 Anti-Nausea Wristbands for Children, or users with smaller wrists)
- One size fits most
- Wristbands are made of cotton and spandex, button is made out of polypropylene latex-free

Instructions for Use: For best results, it is recommended to wear a wristband on each wrist before and during the duration of travel. Precise positioning of the wristband may improve results. 1. Place the wristband over the wrists with the plastic button facing the inside of the wrist. 2. Find the acupressure points on each wrist by applying three fingers across the wrist, starting from the crease at the bottom of the hand. 3. Locate the two tendons that run parallel to each other in the wrist with your index finger. The button on the wristband should be centered here. 4. For best results, wristbands should be worn on both wrists. Note: Replace wristbands when it loses elasticity due to wear over time.

Caution: For external use only – use as directed. Anti-nausea wristbands help provide temporary relief of nausea and vomiting only. If nausea and vomiting is frequent and persistent, seek medical attention right away. Individual results may vary. If local skin irritation, pain or swelling occurs, remove wristbands and rinse area with clean water – consult a pharmacist or physician if symptom persists. This product is not a toy. Keep out of reach of children as button may pose as a choking hazard if removed.

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PharmaSystems Inc.