# KwikDose™ Oral Dispensers and Tip Caps

The KwikDose™ Oral Dispenser is designed with an oral tip to provide an accurate method of measuring and dispensing oral liquid medications. The dispenser's oral tip is designed not to accept or connect with other standardized small bore connections.

## Specifications

- Non-sterile
- ISO 80369-1 compliant
- Meets ISO 7886-1 accuracy requirements
- Oral Tip Caps are self-righting
- KwikDose™ Oral Dispensers are available with both clear and amber barrels
- Not made with natural rubber latex
- Oral Dispenser - Made in Denmark
- Oral tip cap - Made in the USA
- Packaged in Canada

## Material

- Dispenser Barrel: polypropylene (PP)
- Plunger Rod: polypropylene (PP) with polyethylene (PE)
- Plunger Seal: silicone (Si)
- Lubricant: silicone (Si)
- Oral Tip Cap: high-density polyethylene (HDPE)

## Indications for Use

- KwikDose™ Oral Dispenser is intended to administer a measured amount of oral liquid medication.
- Oral tip cap is for use with the KwikDose™ Oral Dispenser to reduce the risk of touch contamination and medication leakage.

⚠ **Contraindications:** None when used as described in product insert.


## ⚠ Precautions

- KwikDose™ Oral Dispensers are for oral use only.
- Verify compatibility between drugs and drug contact materials of this product with the drug manufacturer's package insert and/or material safety documentation.
- Do not reuse KwikDose™ Oral Dispensers or tip caps. Product reuse can be unsafe because of the risk of infection, cross-contamination, endotoxin reaction, patient injury due to material failure and/or chemical reaction due to cleaning or sterilization agents.
- Dispose of properly according to applicable disposal rules and regulations.
- Rapid Squirting of liquid into mouth may cause choking.
- Oral tip caps are a choking hazard and should be kept out of the reach of children.

## Instructions for Use

Any serious incident that has occurred in relation to the device should be reported to the Canadian importer and appropriate regulatory authorities.

1. Ensure the plunger is fully depressed prior to filling the dispenser.
2. Fill KwikDose™ Oral Dispenser to the desired volume.  
Note: volume measurement reference is the edge of the black silicone ring closest to the medication.
3. Apply oral tip cap by pressing it into the dispenser tip.
4. Ensure patient is upright prior to administration.
5. Remove and dispose of the oral tip cap prior to administration.
6. Direct dispenser tip towards the inside of the cheek and slowly depress the plunger to dispense the liquid into the mouth.

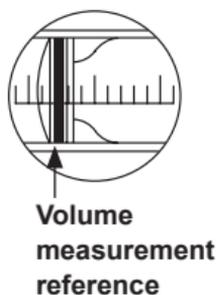Imported/Distributed by:  
PharmaSystems Inc.,  
151 Telson Road  
Markham, ON, L3R 1E7

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