

Hazardous Drugs Spill Kit



Directions For Use:



1. Put on PPE:
Disposable Gloves,
Gown, Shoe Covers,
and Safety Shield.



2. Sprinkle Green-Z®
over spill evenly and
allow liquid to congeal
for safe handling.



3. Remove collected
material with Scoop
and Scraper and place
in **Yellow Waste Bag**.



4. Wipe up any
residue left behind
with supplied Wiper
Pads. **Clean area**
with **surface wipe**
or **disinfectant**.



5. Remove PPE and
place all contaminated
materials into Yellow
Waste Bag. Seal bag
with supplied Twist Tie.



6. Fill out Identification
Tag and attach. **Dispose**
of bag in accordance
with local, state, and
federal regulations.

Contents:

- 21 g Green-Z® Fluid Solidifier
- Caution Sign
- Full-Face Safety Shield
- Identification Tag
- Instruction Card
- Scoop & Scraper
- Shoe Covers, Non-Slip
- Twist Tie
- USP <800> Compliant Gown
- 2 pairs of USP <800> Compliant Nitrile Gloves
- 2 Wiper Pads (11" x 11")
- Yellow Biohazard Waste Bag (20" x 13" x 39")



Buffalo, NY 14215
(800) 456-7077
www.safetec.com

CAUTION!

HAZARDOUS
DRUG SPILL



PROCEED WITH CARE!

See reverse side for directions for use.

