DIGITAL ELECTRONIC BASAL THERMOMETER

INSTRUCTIONS FOR USE - Please read carefully

Features

- Easy to read digital liquid crystal display (LCD).
- Temperatures should be taken under the tongue - for oral use only.
- Auto shut-off after approximately 10 minutes of inactivity.
- ABS resin body presents less hazard than mercury glass thermometers.
- Low Battery Indicator: Replace battery when  appears in lower right corner of display.

Thermometer's measuring range: 32.00~42.90°C
 Resolution: 0.01 °C
 Accuracy: +/-0.05°C, 35.50~42.00°C +/-0.1°C below 35.50°C or over 42.00°C
 Battery: LR 41- Alkaline button cell battery (0.0% Hg)
 Dimensions: 12.4 cm L x 1.8 cm W x 0.9 cm H
 Beeper: Approximately 10 seconds after thermometer reaches a stable temperature reading
 Memory: Built-in memory enabling display of previously measured value.
 Working Conditions: Relative Humidity: 25% ~ 80%, Temperature: 10~35°C
 Storage Conditions: Relative Humidity: 25% ~ 80%, Temperature: -25~55°C

Precautions

1. Do not boil the probe. Clean the unit by wiping it with a dry cloth and disinfect the probe with isopropyl (rubbing) alcohol.
2. Do not drop the thermometer or expose to heavy shock. The unit is not shock proof.
3. Do not bite the probe.
4. Store thermometer at room temperature. Do not store the unit in direct sunlight, at high temperatures, in high humidity or dust.
5. Stop using the thermometer if it operates erratically or if display malfunctions.
6. Keep out of the reach of children and pets. Battery cover and battery may be fatal if swallowed. If this occurs, seek medical attention immediately.
7. Do not attempt to disassemble the unit except to replace the battery (follow instructions below).

Battery Replacement

1. Replace battery when  appears in lower right corner of LCD display.
2. Pull battery cover off.
3. Gently pull out plastic circuit board with battery chamber approximately 1cm.
4. Use a small instrument like a toothpick to remove battery. Avoid using a sharp metal object.
5. Place the new battery in the chamber with positive side (+) up and negative side (-) facing down.
6. Slide battery chamber back into place and battery put the cover on.

Warning: Discard old battery carefully. Do not dispose of the battery in a fire. It may explode.

Instructions for use

1. Disinfect the probe before use.
2. Press the on/off switch. A tone will sound and the display will read 188.8°F.
3. The display will flash "Lo" (°F or °C). The last recorded temperature is displayed before the screen displays back to "Lo" (°F or °C).
4. Position the probe under the tongue to measure the temperature. Close mouth and breathe through the nose.
5. The degree sign on the display flashes throughout the measuring process. 10 seconds after thermometer reaches a stable temperature reading, the flashing will stop and an alarm will beep. The measured reading will appear on the LCD. The displayed reading will not change when the thermometer is removed from its testing position.
6. The unit will automatically turn off in approximately 10 minutes. To prolong battery life, it is best to turn off the thermometer once the temperature has been noted.
7. To check memory, the thermometer must be off. Press and hold the on/off button. Keep the button pressed and after about 2 seconds the last recorded temperature will appear. Once the button is released the self-checking temperature of 36.50 will appear and the thermometer is ready for use.

Note: Normally the buzzes are "Bi – Bi – Bi – Bi". Alarm beeps more rapidly when temperature reaches 37.8°C (100°F) or higher and the buzzes are "Bi-Bi-Bi ---- Bi-Bi-Bi ---- Bi-Bi-Bi". If room temperature is higher than 90°F (32°C) prior to testing, the LCD will display the actual room temperature instead of "Lo" (°F or °C).

In Step 4, you may put a sanitary plastic probe shield on the sensor tip to help prevent cross-infection and recontamination. Use of probe cover may result in a 0.1°C (0.2°F) difference from actual temperature. Dispose of any used probe cover and store the unit in safe place

RELIABLE METHODS TO MEASURE BASAL TEMPERATURE

ORAL USE

Take temperature first thing in the morning while still lying in bed, prior to food, drink, or activity. A minimum of 4-5 hours of sleep is required for an accurate BBT measurement. Record the temperature on the chart provided. Take the temperature at approximately the same time each day. It is advisable to chart your BBT for a couple of months to understand your cycles better.

It is strongly recommended that you consult with your Physician when beginning family planning procedures. They can further assist you with interpreting the information on your tracking chart.

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