

GET YOUR SPILLS UNDER CONTROL WITH OUR **RED-Z** SPILL CONTROL SOLIDIFIERS

Formulated with **Chlorine** to deodorize
blood and body fluid spills.

Red-Z® Spill Control Solidifier

First developed in the late 1980s, Red Z® Solidifier was used to help prevent infection during the AIDS epidemic. Unexpected spills can happen anytime, anywhere – no matter what type of industry. With the use of Red Z®, you can reduce the risks from improper spill cleanup by relying on the solidifying agent to do the hard work for you.



Fast Acting



Deodorizes



Splash Free

Features

- Fast-acting and easy-to-use
- Eliminates splashing by solidifying and turning spills into a semi-solid (gel) mass
- Deodorizes spills (contains chlorine)
- Cost-effective cleanup solution

Directions for Use

1. Wear appropriate protective clothing.
2. Sprinkle Red-Z powder to surround the spill and then evenly distribute over it. Allow sufficient time for solidification to occur.
3. Remove solidified material with scoop and scraper (or similar tool) and place in an appropriate container.
4. Clean and disinfect contaminated areas after use and wash thoroughly.

 **PharmaSystems®**

PharmaSystems.com
151 Telsons Road
Markham Ontario
Canada L3R 1E7



Item No.: 41103

Size: 15 oz.

Case Qty: 12

Expiration Dating

Green-Z® is shelf stable and does not require an expiration date. In order for the product to remain shelf stable, it must be kept in a cool, dry place and in its original packaging. If contents are hard or compact, simply shake the product to loosen, before use.

**1oz solidifies 1L of
water-based fluid**

