



Arisure®

13mm (YM101), 20mm (YM102),
& 28mm (YM054)
Closed Vial Adapters

The Arisure Closed Vial Adapter (CVA) is a safe and easy-to-use vial access device for use with hazardous medications. Its low attachment force and non-coring spike help eliminate problems during vial access. The Arisure CVA is available in 13mm, 20mm, and 28mm sizes. When used with an Arisure Closed Male Luer (CML), the Arisure CVA forms an intuitive, secure Luer lock connection that enables safe, simple transfer of medications. The Arisure CVA remains mechanically and microbiologically closed, preventing leaks and maintaining drug vial sterility for up to 7 days when used and properly stored according to USP <797>, and reducing the risk of contamination.

PRODUCT OVERVIEW

PRODUCT DESCRIPTION AND FUNCTION

The Arisure CVA is a stand-alone, single-use, disposable device that permits access to a medication vial without the use of a needle. All Arisure CVA configurations are designed to mate with standard medication vial enclosures of complementary dimension and feature a needle-free valve for mating Luer access. This device is used for reconstituting and transferring potentially hazardous fluids or other medications.

The Arisure CVA used with the Arisure CML creates a versatile closed system that is used for reconstituting and transferring potentially hazardous fluids or other medications. The system supports dry disconnection, minimizes fluid leakage, prevents the spread of vapor into the working environment, and reduces exposure to harmful drug residue. The Arisure CVA and Arisure Neutral Valve (ANV) are contraindicated for use with blunt cannula systems.

INTENDED USE

Arisure is a Closed System Drug Transfer Device (CSTD) that mechanically prohibits the release of drugs in vapor, aerosol or liquid form during preparation and administration, and prevents the introduction of microbial and airborne contaminants into the drug or fluid path, allowing the system to minimize exposure of individuals, healthcare personnel, and the environment to hazardous drugs.

REGULATORY

US FDA Product Classification	ONB and LHI
Quality management system certified	ISO 13485

PRODUCT CONFIGURATIONS

DEVICE CODE	DESCRIPTION	AVAILABLE FOR SALE	ORDERING INFO	PACKAGING INFO
 YM101 PharmaSystems Part Code: 88101	13mm Arisure CVA with lubricated spike	US	Case Qty: 50	Case dimensions: 23 in x 10.25 in x 8.5 in Gross weight per case: 3.95 lbs
 YM102 PharmaSystems Part Code: 88102	20mm Arisure CVA	US	Case Qty: 50	Case dimensions: 23 in x 10.25 in x 8.5 in Gross weight per case: 4.05 lbs
 YM054 PharmaSystems Part Code: 88054	28mm Arisure CVA	US	Case Qty: 50	Case dimensions: 23in x 10in x 10in Gross weight per case: 4.25 lbs

The information in this sheet is not intended to be exhaustive. The information is liable to modification from time to time in light of experience and our commitment to continuous product development. These documents, in part or in whole, may not be copied or reproduced without prior written permission from Yukon Medical.