



Arisure®

Dry Spike Non-Vented (YM064)

The Arisure Dry Spike (DS) Non-Vented provides an easy, safe way to transfer hazardous medications into a standard intravenous (IV) fluid bag or container and then to your patient. When used with the Arisure Closed Male Luer (CML), the device provides a secure, intuitive connection and a dry disconnection that reduces exposure to harmful drug residue. The Arisure Dry Spike Non-Vented's double seal design allows for secure and safe connections with a variety of IV set spikes. The device's high flow rate and ease of secure attachment to a bag, bottle, or other semi-rigid container provide a safe, closed system transfer of medication to your patient.

PRODUCT OVERVIEW

PRODUCT DESCRIPTION AND FUNCTION

The Arisure Dry Spike Non-Vented is an infusion access device for medication delivery from a bag, bottle, or semi-rigid container to an IV set. The device features the needle-free Arisure Neutral Valve for secure transfer of hazardous medications to IV containers while minimizing exposure to hazardous liquid or aerosols by means of a dry disconnection from the Arisure Closed Male Luer.

Key features of the Arisure Dry Spike Non-Vented include two independent lumens for drug admixture and patient delivery, respectively. The device provides a high flow rate as well as a pierceable septum that creates a dry connection for an administration set.

INTENDED USE

Arisure is a Closed System Drug Transfer Device (CSTD) that mechanically prohibits the release of drugs in vapor, aerosol or liquid form during preparation and administration, and prevents the introduction of microbial and airborne contaminants into the drug or fluid path, allowing the system to minimize exposure of individuals, healthcare personnel, and the environment to hazardous drugs.

REGULATORY

US FDA Product Classification	ONB and LHI
CE marking	CE 2797
Quality management system certified	ISO 13485

PRODUCT CONFIGURATIONS

DEVICE CODE	DESCRIPTION	AVAILABLE FOR SALE	ORDERING INFO	PACKAGING INFO
 YM064 PharmaSystems Part Code: 88064	Arisure Non-Vented Dry Spike	US, CA	Case Qty: 50	Case dimensions: 15.25 in x 5.875 in x 7.75 in Gross weight per case: 2.55 lbs