

VACCINE CLINIC SUPPLIES

2026 CATALOGUE



Quality Supplies and Exceptional Service

Proudly Canadian owned and operated since 1977, PharmaSystems is your Single Source for Pharmacy Supplies.

We have a Medical Device Establishment License for Class 1 and Class 2 devices, registered with Health Canada, and we are ISO Certified –13485:2016 MDSAP. We proudly support a wide range of healthcare & institutional partners across Canada.

We have created this curated catalogue to highlight our most-requested supplies to support vaccine clinics.

Visit our full catalogue online:
PharmaSystems.com



How to order:

Online: www.pharmasystems.com
(register your account for access)

Call: 905.475.2500 (Toll free: 1.888.475.2500)

Fax: 905.475.7155 (Toll free: 1.888.475.7155)

Email: orders@pharmasystems.com

Our bilingual customer service team is readily available to help!

Office Hours: Monday-Friday
8:30am-1:00pm, 1:30-4:30pm, EST.

PharmaSystems Inc. 151 Telson Road
Markham Ontario Canada L3R 1E7

Shipping Terms: Orders will be shipped to all destinations in Canada. We reserve the privilege of shipping via the carrier of your choice.

Prices: Prices are subject to change without notice. Login to our website portal to view all catalogue pricing.

Returns: All returns to PharmaSystems must be authorized, and subject to a 25% restocking fee.

Accessibility: In meeting the needs of our customers and employees we strive at all times to provide our products in a way that respects the dignity of all individuals including persons with disabilities. In order to ensure equal treatment all PharmaSystems' customer service representatives and administrators are trained on policies, practices and procedures with respect to our commitment to people with disabilities.

Contents

Vaccination Products

- 4 Safety Needles
- 4 Luer Lock Syringes

Protective Apparel

- 5 Protective Apparel
- 7 Dispensers & Organizers

Infection Control

- 8 Sharps Disposal Containers
- 9 Biohazard Bags
- 9 Emergency Spill Kits
- 10 Sterile Alcohol
- 10 Wipes

Temperature Monitoring & Storage

- 13 Traceable® Thermometers
- 16 StatTemp Monitoring Solution
- 17 Refrigerators
- 18 Locking Refrigerator Storage
- 21 Exposure Indicators
- 21 Insulated Transport

Storage & Organization

- 22 Bins, Lids & Dividers
- 25 Mesh Baskets
- 25 Tilt Front Bins

General Pharmacy Supplies

- 26 Rx Fill Trays
- 26 Prescription Boxes & File Folders
- 27 Hanging Prescription Bags
- 27 Prescription Receipt Pads
- 28 Tape & Dispensers
- 28 Prescription Labels

30 Labels

- 29 Storage Information Labels
- 29 Pharmacy Consultation Labels

Order Forms

- 30 Vaccine Clinic Essentials – Quick Order Form
- 31 Custom Label Order Form
- 32 Blank Order Form

Online Account Benefits:


- ✓ Competitive pricing
- ✓ 1000's of in-stock supplies ready to ship
- ✓ Quick inventory replenishment & same-day order processing
- ✓ Consolidate orders through a single source for supplies
- ✓ Shipping from our local warehouse, delivery anywhere in Canada



Sol-M® Luer Lock Syringes

- Low Dead Space (LDS) syringes minimize “dead space”, which is the area within the syringe that retains medication even after it has been injected. Minimizing this “dead space” improves the efficiency of medication delivery by reducing the amount of medication wasted with every administration
- LDS Syringes increase the number of doses from each vial
- High-transparency barrel to easily visualize contents
- Silicone lubrication ensures smooth plunger movement
- Luer lock syringe available in a wide range of sizes and connection combinations
- Safe plunger backstop prevents accidental withdrawal of plunger
- Individually packaged


Item #	Description	Qty
V0201	Sol-M® 1ml Low Dead Space Luer Lock Syringe (without needle)	100
V0203	3ml Luer Lock Syringe (without needle)	100

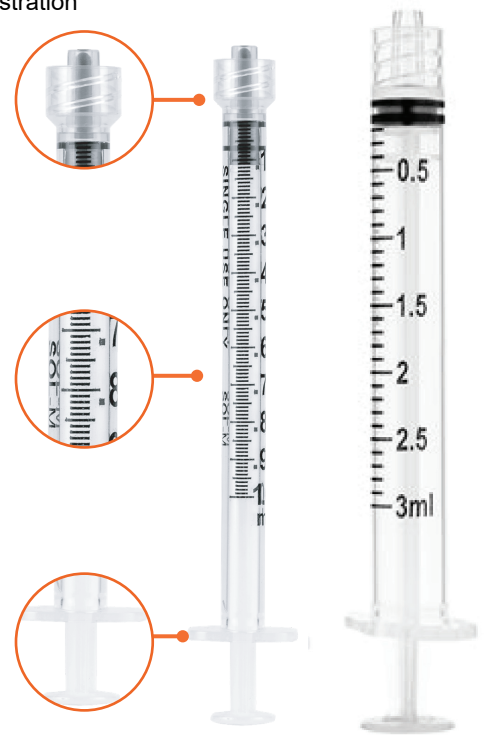

Sol-Care® Safety Needle

- Sol-Care™ Safety Needle is a single use, standard hypodermic needle with a protective arm that is used to enclose the needle after use to protect the healthcare professionals from needlestick injury
- In the activated position, the protective arm guards against accidental needle stick during normal handling and disposal of the used needle/syringe combination
- Two options for protective arm activation: hard surface activation and thumb activation
- Simple one-handed safety shield mechanism
- Individually packaged

Item #	Description	Qty
V0205	Adult Needle 1" (Adult)	100
V0206	Safety Needle 1-1/2" (For Higher BMI Population)	100
V0207	Safety Needle 5/8" (Adolescents/Infant)	100

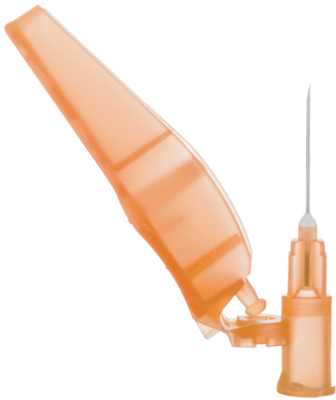

Hard Surface Activation



Thumb Activation



Beard Guards

- Ideal to cover beard, mustache, and sideburns
- Comfortable elastic strap loops behind ears
- Universal size
- Material - Polypropylene

Item #	Colour	Qty
12077-31	Blue	100



Plastic Shoe Covers

- Ideal for electronics processing, food processing, hospitals, laboratories, inspections and maintenance
- Elastic on top for easy usage and removal
- Textured bottom for skid resistance
- Flexible material for easy movement
- Made out of non-woven polypropylene

Item #	Size	Qty
S741	Universal	500 pairs

Disposable Isolation Gown

- Disposable gown meets AAMI PB70 Level II barrier protection
- Gown made of non-woven material with full back closure.
- Features: Flat belt, elastic cuffs and tie-on neck closure.

Item #	Size	Qty
GW100Y	L	10
GW10Y	L	100



Personal Protection Gown

- Disposable gown has a stirrup cuff design to give the wearer added protection to the wrist area
- Back panel is open for cool comfort during long periods of wear
- Slip-on style neck loop (the shoulder panel at back is perforated for quick and easy removal)
- Made of embossed polyethylene, latex-free that passes ASTM F1670-95 standards

Item #	Size	Qty
20497	Universal	50

Nitrile Medical Examination Gloves

- Textured for extra grip and dexterity
- Excellent tensile strength and touch sensitivity
- Specify size when ordering

Item #	Sizes	Colour	Thickness	Qty
GNSMS	S	Blue	4 mil	100
GNSMM	M	Blue	4 mil	100
GNSML	L	Blue	4 mil	100



Digital Thermometer, °C/°F

- For axillary or oral use only
- Large and easy to read liquid crystal display
- Fast and accurate - 10 seconds read time

Item #	Description
PS720 A	Flex Tip Digital Thermometer
PS730 B	Digital Thermometer

Surgical Masks

- Lightweight and easy to use
- Elastic ear loops
- Pleated design for maximum comfort

Item #	Quantity
PS345 C	10
B344	50

Universal Face Mask Box Holder

- Versatile - Securely holds a single box of face masks in a wide variety of sizes.
- Compatible - Works with boxes no larger than 8"W x 5½"H x 5½"D and no smaller than 6½"W x 4"H x 4½"D.
- Convenient - Can mount to most smooth surfaces using keyholes on the back.
- Material - Beige ABS plastic
- Dimensions - 8¼"W x 4"H x 5¾"D

Item #
20336 D

Reusable Face Mask

- 2 per package
- Material: Double layer fabric
- Not to be shared. Single person use only

Item #	Description
B341 E	Junior mask Size

Face Mask Ear Relief Bands

- 10/package
- Size: 8⅝"L x 1⅛"H

Item #
20445 F



Glove Box Holders

- Conveniently stores multiple boxes in one location
- Store several sizes or different types of gloves in one holder
- Designed with key holes for mounting
- Can be wiped clean with soap and water or hospital sanitizer
- Stainless steel designs are smart choices for USP 797 compliance

Item #	Box Capacity	Description	Dimensions (W x H x D)	
7468-01	A	2	White Enamel	10-7/8 x 10 x 4"
7466-01		2	Clear Plexiglas	10-7/8 x 10 x 4"
7474-01		3	Stainless Steel	16-1/8 x 10 x 4"
7461-01	B	3	White Enamel	16-1/8 x 10 x 4"
7467-01		3	Clear Plexiglas	16-1/8 x 10 x 4"
11948	C	4	Stainless Steel	22-1/4 x 10 x 3-3/4"
11949		4	White Enamel	22-1/4 x 10 x 3-3/4"
11950		4	Clear Plexiglas	22-1/4 x 10 x 3-3/4"

Wire Glove Box Holders

- Stainless steel holder with tough white plastic coating
- Can accommodate a variety of glove box sizes
- Open design allows for carton copy to be readable

Item #	Box Capacity	Description	Dimensions (W x H x D)	
8245	D	3	Horizontal	11 x 14 x 4
7430-01	E	1	Horizontal	11.25 x 4.25 x 3.5

All Purpose Bulk Dispenser, Single Compartment

- Provides bulk storage for gloves, bouffant caps, shoe covers, hair nets, etc., to help reduce restocking and keep supplies stored in high-use areas
- Hinged lid provides dust protection and flips up for easy refill
- Dispensing window in the lower front allows quick access
- Place on counter or mount to most surfaces using the two key hole slots on the back panel

Item #	Material	Dimensions (W x H x D)
7426-01	F PETG	12-1/4 x 17 x 6"

Sanitizing Station

- Encourages infection prevention by providing protection supplies in waiting areas
- Clear PETG plastic - easily view inventory
- Hang using the keyhole slots or place on a counter or desktop
- Non-skid feet
- Can hold up to two earloop style face mask boxes, two tissues boxes or one box of gloves and a hand sanitizer bottle

Item #	Dimensions (W x H x D)
10383	G 15 x 8-1/4 x 5-3/4"



ChemoSafety™ Sharps Disposal Containers

- Designed for sharps disposal as well as other types of chemotherapy waste, including IV bags, tubing, and syringes
- Container features snap-on lid with locking rotor that locks for final disposal
- Sealing gasket guards against spills - a pad on the bottom of the container absorbs excess liquid
- Labelled "Chemotherapy Waste"
- Nest unused containers inside each other for stacking
- Made of polypropylene plastic - dispose by incineration

Item #	Capacity	Dimensions (W x H x D)	Qty
9605-10 A	2 gallon	10-1/8 x 9-1/2 x 6-3/4"	20
9607-01 B	8 gallon	15 x 17 x 10-1/4"	1
9607-10	8 gallon	15 x 17 x 10-1/4"	10

SharpStar™ Sharps Containers

- Minimize the potential for over-filling by stopping in the "full" position
- The "always-open" lid is ready for quick disposal
- Large opening easily accommodates 60 mL syringes
- Extended flap keeps hands away from the container when locking
- Tamper resistant opening
- Smart choice for USP 797 compliance
- Complete system (item # 9600-01) includes a wall enclosure with lock, large glove dispenser, a 5 qt. (4.73 liters) transparent red sharps container and an in-service sticker

Item #	Capacity	Dimensions (W x H x D)	Qty
9601-01 C	5 quart	10-3/8 x 9-1/8 x 3-3/4"	1
9601-06	5 quart	10-3/8 x 9-1/8 x 3-3/4"	20

Biohazard Sharps Disposal Containers

- Labelled "biohazard"
- Nest unused containers inside each other for stacking
- Complies with OSHA regulations for handling sharps
- 0.95 L (Item # 9604-01 and 9604-10) has a clear hinged lid with a variety of openings in different sizes for disposal of needles, lancets, butterfly tubing, and small syringes
- Made of polypropylene plastic - dispose by incineration

Item #	Capacity	Dimensions (W x H x D)	Qty
9602-01 D	2 gallon	10 x 9-3/4 x 6-7/8"	1
9602-06	2 gallon	10 x 9-3/4 x 6-7/8"	20
9603-01 E	8 gallon	15 x 17 x 10-3/8"	1
9603-04	8 gallon	15 x 17 x 10-3/8"	10
9604-01	1 quart	3-3/4 x 6 x 3-3/4"	1
9604-10 F	1 quart	3-3/4 x 6 x 3-3/4"	100



Biohazard Leakproof Bags

- Closure - Adhesive closure with perforated tear zone.
- Special Feature - When closed correctly, bags are 100% leakproof bags and include a document pouch for important paperwork.

Item #	Thickness	Dimensions (W x H)	Qty
20426	2 mil	6 x 9"	100



Biohazard Bags


- Meets performance criteria established by OSHA
- Small (Item # 7446) has a fold over flap for closing
- Can be secured closed with a heavy duty security seal with prongs (see Item # 7836)

Item #	Thickness	Dimensions (W x H)	Qty
7446	A 11/2 mil	11-1/4 x 14-1/2"	50
7445	B 4 mil	20 x 39 x 121/4"	25

Stick-On Biohazard Bag

- Includes an adhesive strip to stick to carts, cabinets, walls, etc.
- When bags are full, the top flap folds over and a second adhesive safely seals the bags

Item #	Thickness	Dimensions	Qty
9938	2 mil.	12" W x 14" H	100

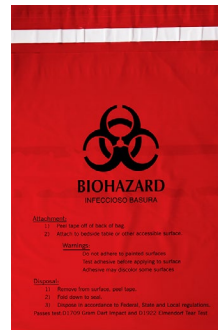

Economy Emergency Spill Kit | 7464-01

- Perfect for small spills that need immediate attention
- Intended for one time use only

Kit Includes

- 1 Bio-Wipe™ Bag
- 1 pair of latex free nitrile gloves
- 13.75 x 17.75" waffle-design paper towelette
- 1 antimicrobial hand wipe
- 2-ounce spray pump bottle of Cavicide® Cleaner
- Instruction sheet



Sterile Alcohol Spray Bottles

- **Compliance:** ISO Class 5 glove and surface cleaning
- **Use:** Wipedown of isolators, cabinets, laminar flow hoods and surfaces
- Use on gloved hands in sterile suites
- Wipe down of products and equipment for pass through to controlled environments
- Ideally suited for use with sterile cleanroom wipes
- **Active ingredients:** 70% by volume USP Grade Isopropanol (IPA) and 30% USP Grade Purified Water, filtered to 0.2µ
- **Features:** sterile alcohol spray, induction sealed to assure stability



Item #	Description	Capacity
17979	A Sterile alcohol spray bottle	16 oz.
18627	Sterile alcohol spray bottle	32 oz.
20254	B Sterile Isopropanol Alcohol, Flip-Top	16 oz

Amplitude™ Wipes

- **Use:** Ideal for spill kits and cleaning carts
- **Material:** Lightweight spunlaced cellulose/polyester
- **Features:** Economical option for general cleaning

Item #	Dimensions	Qty
18958	9 x 9"	300



Lint-Free Wipes

- **Compliance:** USP <797> equipment cleaning guidance standards
- **Use:** Ideal for cleaning lab equipment and medical instruments
- **Features:** Lint-free – no traces of fibre or residue


Item #	Dimensions	Qty
18699	4 x 4"	300
17509	9 x 9"	300
17503	12 x 12"	150

EcoClean Wipes®

- **Compliance:** ISO Class 5 (class 100)
- Use:** Designed for critical care units and cleanrooms
- **Material:** 100% natural fibre
- **Features:** Environmentally friendly – 100% biodegradable
- Low-linting, highly-absorbent, durable
- Heat-resistant, autoclavable
- Quarter-fold design promotes proper wiping technique

Item #	Dimensions	Qty
17883	5 x 5"	100 sheets
17884	9 x 9"	300 sheets
17885	12 x 12"	150 sheets



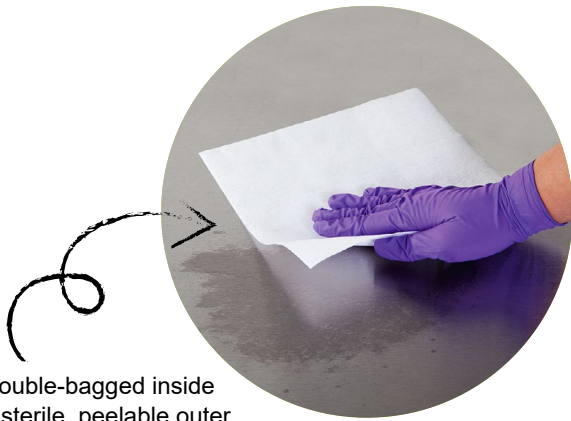

Double-bagged inside a sterile, peelable outer bag and linear tear inner bags, ready to introduce in cleanroom

SterileSorb™ Wipes

- **Compliance:** ISO Class 5
- **Use:** Achieve USP <797> compliance
- Ideal for general purpose cleaning in critical environments: cleanrooms, labs, and compounding pharmacy areas
- **Material:** Nonwoven, cellulose/polyester
- **Features:** Sterile
- Extremely absorbent – low in extractable residues and particles for maximum efficiency in cleanroom settings
- Ideal choice for spill kit packaging

Item #	Dimensions	Qty
18956	9 x 9"	25
18957	12 x 12"	25

Pro-Wipe® All-Purpose Low Linting Wiper, Perforated

- 250 sheets per pack; 4 packs per case
- Durable and low linting with highly absorbent cellulose core
- Excels at absorbency of water and oil based solutions
- **Material:** 50% Rayon 50% Polyester

Item #	Dimensions
17783	9 x 15"



Presaturated Cleanroom Wipers

- **Compliance:** ISO Class 5 and above
- **Use:** Ideal for general-purpose cleaning, surface preparation or wipe down of articles during pass-through. Also effective for residue removal after detergent or disinfectant use
- **Material:** Non-woven, cellulose/polyester
- **Saturated with:** 70% semiconductor grade isopropyl alcohol (IPA) and 30% deionized water (DI)
- **Features:** Ensures consistent saturation and improved control of cleaning protocols
- Excellent strength and durability
- Higher saturation level to support critical solution contact time needed
- Peel and reseal pouches allow for single-wipe use and prevents solvent evaporation
- Lot number printed on each package for traceability and process accountability

Item #	Dimensions	Qty
17880	9 x 9"	75

PharmaSystems' Temperature Monitoring Solutions

Our product comparison chart highlights key temperature monitoring & data logging features: *The National Vaccine Storage and Handling Guidelines (2015)* as well as many provincial standards outline requirements for refrigerator and freezer temperature monitoring. Refer to the full guidelines, or contact us for assistance.



Thermometers with glycol-filled bottle probes

- Recommended based on the National Vaccine Storage and Handling Guidelines (2015)
- 10 ml & 5 ml glycol bottle probes insulate sensors from rapid temperature changes, ideal for mimicking vaccine temperatures



Thermometers with enclosed temperature buffered ambient sensor probes are ideal for:




















- Freezers
- Water baths
- Refrigerators
- Heating blocks
- Incubators

Vaccine Temperature Monitoring Requirements:

- Manual documentation of vaccine temperatures, where the: current, minimum and maximum refrigerator temperatures, and the ambient room temperature must be recorded at the beginning and end of each day (min/max temperatures must be reset after each recording) – log templates are available
- The digital display should be placed on the outside of the refrigerator (Ontario requires a display with a minimum resolution of 0.1°C), with the probe in the centre of the refrigerator (preferably in a vaccine box)
- Bottle probes (bio-safe glycol fluid bottle probes) are recommended for a more accurate reading
- Reset function required - The digital thermometer must be reset after each properly recorded temperature for useful readings
- Thermometers should be set to alarm if temperatures fall or rise to specific values to signify approaching out of range
- Thermometers should be checked for calibration at least annually, and it is recommended they be calibrated within ±0.5°C (±1°F) of accuracy
- Thermometers should have a low battery indicator, and batteries should be replaced every 6 months to 1 year

General Vaccine Storage Requirements:

- Maintain required vaccine storage temperatures
- Hold sufficient inventory
- Have a calibrated temperature-monitoring device inside each storage compartment
- Refrigerator is dedicated to the storage of vaccines only
- Be placed in a secure location away from unauthorized and public access
- Recommended thermometer calibration should be accurate within ±0.5°C (±1°F)
- Record refrigerator temperature twice daily
- Contact the local public health office or immunization program if vaccines are exposed to temperature outside the recommended range

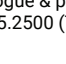Item #	Description	5 ml Probes	10 ml Probes	Ambient Probes	Resolution	Accuracy	Current/Min/Max Temp. (with reset button)	Ambient Room Temp.	High/Low Alarm	Calibration Certificate	Low Battery Icon
10364	 TraceableLive® Wifi Datalogging Traceable® Thermometer		✓		0.01°C	±0.25°C	✓		✓	✓	✓
14140	 Traceable® Refrigerator/Freezer Thermometer		✓		0.1°C	±0.5°C	✓	✓	✓	✓	✓
14141	 Traceable® Traceable® Digital Humidity/Temperature Meter				1°C	±1°C				✓	
14144	 Traceable® Refrigerator/Freezer Thermometer	✓			1°C	±1°C	✓	✓	✓	✓	✓
10366	 Traceable® Dual Thermometer			✓	1°C	±1°C	✓	✓	✓	✓	✓
10367	 Traceable® Refrigerator/Freezer Thermometer			✓	1°C	±1°C	✓	✓	✓	✓	✓
10368	 Traceable® Refrigerator/Freezer Thermometer		✓		1°C	±1°C	✓	✓	✓	✓	✓
10369	 Traceable® Dual Bullet Probe Thermometer			✓	0.1°C	±1°C	✓	✓	✓	✓	✓
17751	 Traceable® Jumbo Thermometer		✓		0.1°C	±1°C	✓	✓	✓	✓	✓
17752	 Traceable® Jumbo Thermometer	✓			0.1°C	±1°C	✓	✓	✓	✓	✓
17750	 Traceable® Jumbo Thermometer			✓	0.1°C	±1°C	✓	✓	✓	✓	✓
17988	 Traceable® High-Accuracy Thermometer		✓		0.01°C	±0.3°C	✓		✓	✓	✓
17989	 Traceable® High-Accuracy Thermometer, dual probe		✓		0.01°C	±0.3°C	✓		✓	✓	✓
17990	 Traceable® High-Accuracy Thermometer, dual probe			✓	0.01°C	±0.3°C	✓		✓	✓	✓
19195	 Traceable® Memory-Loc™ Datalogger, single probe		✓		0.01°C	±0.25°C	✓		✓	✓	✓
19196	 Traceable® Memory-Loc™ Datalogger, dual probe		✓		0.01°C	±0.25°C	✓		✓	✓	✓
19197	 Traceable® Excursion-Trac™ Datalogger, single probe		✓		0.01°C	±0.25°C	✓		✓	✓	✓
19198	 Traceable® Excursion-Trac™ Thermometer, dual probe		✓		0.01°C	±0.25°C	✓		✓	✓	✓
19465	 Fridge-Tag® Traceable® Storage Temperature Monitor with USB			✓	0.1°C	±0.5°C	✓		✓	✓	✓

Traceable® Thermometers:

- Triple display simultaneously shows the current, minimum and maximum temperature reading
- Includes Traceable® certificate from ISO 17025 calibration laboratory, accredited by A2LA, valid for 1 year - recalibration is the customer's responsibility, contact us for assistance
- Alarm alerts when temperature rises or falls outside set point
- Low battery symbol indicates when batteries should be replaced
- Reads temperatures ranging from - 50° - 70° C (-58° - 158° F)
- Includes all necessary batteries and mounting equipment (flip stands, wall mounts, Velcro® and magnet strips)



Item #10369 is a dual bottle probe extension for item #10366

Traceable® Thermometers

- Reads the ambient room temperature as well as the external probe temperature using the in/out function
- Alarm programmable in 1° increments

Item #	Description	Probe Type	Dimensions (W x H x D)
14144	A Traceable® Refrigerator/Freezer Thermometer	5 mL Bottle Probe	2-3/4 x 4-1/4 x 3/4"
10367	B Traceable® Refrigerator/Freezer Thermometer	Ambient Probe	2-3/4 x 4-1/4 x 3/4"
10368	C Traceable® Refrigerator/Freezer Thermometer	10 mL Bottle Probe	2-3/4 x 4-1/4 x 3/4"
17751	D Traceable® Jumbo Refrigerator/Freezer Thermometer	10 mL Bottle Probe	3-7/8 x 4-3/8 x 7/8"
17752	Traceable® Jumbo Refrigerator/Freezer Thermometer	5 mL Bottle Probe	3-7/8 x 4-3/8 x 7/8"
17750	E Traceable® Jumbo Refrigerator/Freezer Thermometer	Ambient Probe	3-7/8 x 4-3/8 x 7/8"
14140	F Traceable® High Resolution Refrigerator/Freezer Plus™ Thermometer	10 mL Bottle Probe	3-1/4 x 2-1/5 x 5/8"
10366	G Traceable® Dual Thermometer	2 10 ml Probes	3-1/2 x 1-3/4 x 1"
10369	Traceable® Dual Bottle Probe Extension for #10366	2 Ambient Probes	3-1/2 x 1-3/4 x 1"



Traceable® High-Accuracy Refrigerator/Freezer Thermometer

- Displays the exact time and date when the minimum and maximum temperature occurred
- Displays the exact time and date alarm is triggered
- Alarm programmable in 0.1° increments

Traceable® Excursion-Trac® Datalogging Thermometer

- Datalogging thermometer with user-defined time intervals
- Stores up to 10 unique alarm events - alarm tracking stores each alarm event, the time and date of occurrence
- Rolling memory structure maintains the most recent 525,000 observations, clear after downloading to USB flash drive (unit can remain in use while downloading data)
- Allows recorded data (CSV file) to be transferred from thermometer to PC or Mac using a USB flash drive (not included). No software is needed for computer interface. Each thermometer has a unique ID allowing multiple units to be used in the same location
- Status indicators show low battery, memory full, USB data transfer and active alarm state

Traceable® Memory-Loc™ Datalogging Thermometer

- Can be used in 21 CFR 11 environments – data is locked and cannot be cleared or changed on unit
- Fixed one minute logging intervals, for up to 2 years
- Stores up to 10 unique alarms - alarm tracking stores each alarm event, the time and date of occurrence
- Allows recorded data (CSV file) to be transferred from thermometer to PC or Mac using a USB flash drive (not included). No software is needed for computer interface. Each thermometer has a unique ID allowing multiple units to be used in the same location
- Status indicators show low battery, memory full, USB data transfer and active alarm state

Item #	Description	Probe Type	Dimensions (W x H x D)
17988	A Traceable® High-Accuracy Refrigerator/Freezer Thermometer	10 mL Bottle Probe	2-3/4 x 4-1/4 x 3/4"
17989	B Traceable® High-Accuracy Refrigerator/Freezer Thermometer	10 mL Bottle Probes (2)	2-3/4 x 4-1/4 x 3/4"
17990	C Traceable® High-Accuracy Refrigerator/Freezer Thermometer	Ambient Probes (2)	2-3/4 x 4-1/4 x 3/4"
19197	D Traceable® Excursion-Trac® Datalogging Thermometer	10 mL Bottle Probe	2-3/4 x 4-1/4 x 3/4"
19198	E Traceable® Excursion-Trac® Datalogging Thermometer	10 mL Bottle Probes (2)	2-3/4 x 4-1/4 x 3/4"
19195	F Traceable® Memory-Loc™ Datalogging Thermometer	10 mL Bottle Probe	2-3/4 x 4-1/4 x 3/4"
19196	G Traceable® Memory-Loc™ Datalogging Thermometer	10 mL Bottle Probes (2)	2-3/4 x 4-1/4 x 3/4"

Refrigerator/Freezer Thermometer | 3010

- Temperature Range - From -40°F to 120°F, -40°C to 50°C
- Display - Non-toxic, mercury-free thermometer
- Dimensions - 1¼"W x 4½"H x ⅜"D



TraceableLive® Wifi Datalogging Thermometer | 10364

- Wireless Temperature Monitoring and more—connect to existing WiFi
- Alerts: temperature alarm, connectivity interruption, low battery
- Cloud-based data interface—no local software required; web, Android, Apple IOS data visibility
- Continuous data transmission and monitoring using SSL encrypted SECURE connection
- Receive MOBILE PUSH / E-MAIL / TEXT notifications
- Unlimited scalability—use TraceableLIVE® in individual labs or an entire network of facilities

Item #	UPC
10364	063636103644

Fridge-Tag® Storage Temperature Monitor with USB | 194465

- Fridge-Tag 2L's direct display feature requires no software to quickly review 30 day's worth of data instantly.
- Monitors the temperature inside the unit to protect the integrity of temperature-sensitive drugs
- Dimensions - 5¼"W x 2½"H x ⅝"D



Stainless Steel Refrigerator/Freezer & Dry Storage Thermometer

- Colour zones on dial indicate safe ranges and HACCP guidelines
- Reads temperatures from -30° - 30° C (20° - 80° F), with an accuracy of ±1°C
- Features attached hook for easy hanging or use base for free standing
- NSF certified

Item #	Measures
16087	2" diameter

Traceable® Long Probe Thermometer

- Reads temperatures from -50° – 300° C (-58° - 572° F), 0.1° resolution, accuracy of ±0.1°C
- Reading is updated every second
- Memory recalls previous min./max. readings
- Waterproof and shockproof - stainless steel probe is impervious to acids, bases, and solvents; can be inserted in a variety of solutions
- Includes Traceable® certificate to verify traceability standards
- Battery included (1.5V silver-oxide)

Item #	Dimensions (L x W x H)
10375	10-1/4 x 1-3/4 x 8" Long Probe



Stat Temp 3.0 NIST Certified Refrigerator / Freezer Sensor

- Encased Sensor - Sensor (21266 includes one sensor; 21268 includes two sensors) is housed inside an unbreakable silicone case that can be placed within the wires on most refrigerator and freezer shelves to minimize obstructions.
- Data Logging - Wi-Fi-enabled temperature monitor (included) offers unlimited readings with user-selected logging rate from every 5 minutes up to every 30 minutes.
- Note: Monitors work with 2.4GHz Wi-Fi networks only.
- Mesh-Enabled Connectivity - Devices communicate with each other to extend coverage and improve network reliability, even in challenging hospital layouts. This allows sensors to be installed in distant or obstructed areas without requiring direct Wi-Fi access.
- Set-Up - Once connected to Wi-Fi, the unit can be added to the Stat Temp website, stat-temp.io. The sensor will automatically be added to the dashboard.
- Mobile-Friendly Reports - Cloud-based storage allows instant access to reports from a computer, tablet, phone or other connected device for process, compliance or inspection.
- No Fees - There is no subscription fee for Stat Temp.
- No Limits - Users have unlimited cloud storage and can have multiple Stat Temp monitors linked to their portal.
- Online Portals - Create portals on the Stat Temp website for locations; view all units being monitored or sort to show less (ex. location only).
- Permissions - Set user-defined permissions based on shifts or roles.
- Alerts - Configure real-time and user-defined alerts via text, email or phone call with forced acknowledgment of solutions. Example: Call Calvin, Fred and Cindy
M-F, 8 a.m. – 5 p.m.; email Annie and text Tim
M-F, 5 p.m. – 8 a.m.
- Power Source - Uses a standard USB-C plug. No batteries to change out.
- Loss of Power - 3-day battery backup
- Min/Max Log - Provides min/max data for the last 24 hours.
- NIST - Includes a 4-year NIST certificate.

Item #	Description	Temperature Range	Temperature Accuracy	Dimensions (W x H x D)
21266	Single Sensor	-50°C to 50°C (-58°F to 122°F)	+/-0.2°C	4" x 2½" x 1"
21268	Double Sensor	-50°C to 50°C (-58°F to 122°F)	+/-0.2°C	4" x 2½" x 1"
21270	Ambient Sensor	-40°C to 85°C (-40°F to 185°F)	+/-0.2°C	4" x 2½" x 1"
21272	Ultra-Low Sensor	-100°C to -50°C (-148°F to -58°F)	+/- 0.5°C	4" x 2½" x 1"



Undercounter Freestanding Refrigerator

- Units feature a digital microprocessor temperature controller allowing for the precise temperature management necessary for critical samples and supplies.
- A forced air directional refrigeration system provides superior temperature uniformity and recovery after door openings.
- Environmentally friendly, natural hydrocarbon (HC) refrigerants (not containing hydrofluorocarbons) vastly reduce global-warming potential (GWP) while meeting new EPA/SNAP mandates and UL, ASHRAE compliance guidelines.

Includes:

- Temperature controller
- Digital setting accuracy
- Precise temperature maintenance
- Monitoring and alarms
- High/low temperature alarm
- Sensor error alarm
- Alarm mute
- Remote contacts



Item #	Electrical	Volume	Dimensions (W x D x H)	Cycle	Door Type	No. of Shelves
REF303	115V, 60 Hz, 0.90A	71 L (2.5 cu. ft.)	17-3/4 x 19-1/2 x 28-7/8"	Defrost	1 Glass Swing	4 Adjustable
REF304	A 115V, 60 Hz, 0.90A	51 L (1.8 cu. ft.)	17-1/4 x 19-1/4 x 21-1/4"	Defrost	1 Glass Swing	2 Adjustable




Freestanding Refrigerators

- Designed to enhance pharmaceutical storage and display experience to the highest standards of quality and performance.
- Self contained Cassette® forced-air refrigeration employs R290 hydrocarbon Natural
- Refrigerant within a sealed capillary tube system.
- Cassette® refrigeration is bottom-mounted and removable from the front for ease of maintenance. Refrigeration may be exchanged or upgraded without mechanic. Designed to hold 36°F to 46°F (2.2°C to 7.7°C).
- Equipped with Energy Efficient Digital Controller featuring visual/audible product protection alarm.
- Technologically advanced ProductTemp™ provides exterior product temperature display. Displays to a resolution of 0.1°.


Item #	Electrical	Volume	Dimensions (W x D x H)	Door Type	No. of Shelves
REF12C	B 115V/3.1 Amp/60 Hz	28 cu. ft	24 x 23-7/8 x 62"	1 Glass Swing	4
REF18C	C 115V/3.1 Amp/60 Hz	35 cu. ft	24 x 23-7/8 x 78"	1 Glass Swing	7
REF46H	D 115V/4.3 Amp/60 Hz	78 cu. ft	47-1/2 x 31 x 78"	2 Glass Swing	10

Stainless Steel Locking Refrigerator Storage Boxes

- Ideal solution for areas that cannot accommodate full-sized locking refrigerators self-locking nuts and bolts to refrigerator shelves
- Featured pre-drilled holes to use with security seals
- All locks (except padlocks) come keyed differently on a master key system; contact us for keyed alike options
- Select from styrene drawers, or acrylic (to view contents, noted below)
- Outside box measures: 5¼"W x 4¼"H x 11¼"D
- Inside box measures: 5⅜"W x 3"H x 10⅞"D
- Made of 18-gauge stainless steel
- **Note:** Acrylic drawers cannot be used with security seals or dividers

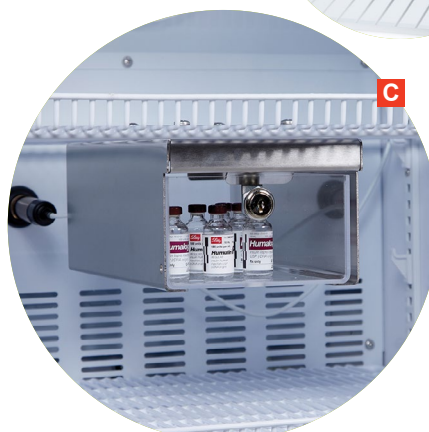

Item #	Description
3725	A Key Lock
3726	B Slam Lock or Padlock
3735	C Acrylic drawer with key lock

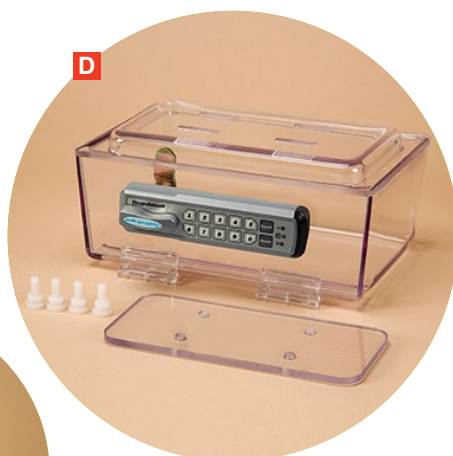

Deluxe Refrigerator Boxes


- Secure - Easily organize and store temperature-sensitive medication in these lockable, tamper-proof refrigerator boxes
- Durable - PETG construction means boxes have superior impact qualities for years of everyday use
- Prevents Rolling - Roll stop lip prevents contents from rolling beyond the edge of the box.

Item #	Description	Dimensions (W x H x D)
17844	D Deluxe Compact Refrigerator Box, KEDL	8-1/8 x 3-1/2 x 4-13/16"
17524	E Deluxe Refrigerator Box, Key Lock	15-3/16 x 3-1/2 x 7-5/8"
17526	F Deluxe Compact Refrigerator Box, Double Key Lock	8-1/8 x 3-1/2 x 41-3/16"

Rugged Refrigerator Boxes

- Secure - Easily organize and store temperature-sensitive medication in these lockable, tamper-proof refrigerator boxes
- Durable - Boxes are made of rugged polycarbonate material that provides extra durability for years of everyday use.
- Prevents Rolling - Roll stop lip prevents contents from rolling beyond the edge of the box.
- Maintain storage capacity - No tools required for assembly. Because there is no obtrusive mounting fixtures, the boxes take virtually no space away from the shelves above them
- Dimensions - 10 $\frac{1}{16}$ "W x 3 $\frac{7}{8}$ "H x 5 $\frac{1}{4}$ "D

Item #	Description
18907	A Rugged Refrigerator Box, Key Lock
18908	Rugged Refrigerator Box, KEDL
18909	B Rugged Refrigerator Box, Dial Combination
19166	Rugged Refrigerator Box, Key Lock, Amber
19167	C Rugged Refrigerator Box, KEDL, Amber

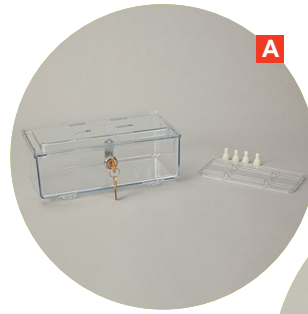

Locking Refrigerator Boxes, Tilting & Non-Tilting

- Secure - Acrylic locking refrigerator boxes organize and secure temperature-sensitive medications.
- Multiple Lock Styles - Available in two locking styles: Key Lock or Keyless Entry Digital Lock.
- Dimensions - 12 $\frac{3}{8}$ "W x 4 $\frac{3}{8}$ "H x 12 $\frac{3}{16}$ "D

Item #	Description
18541	D Tilting Refrigerator Box, Key Lock
18544	Non-Tilting Refrigerator Box, Key Lock

Locking Refrigerator Storage Boxes

- Ideal for use in facilities without access to locking refrigerators
- Ideal for controlling substances under single or double locks
- Lids can be affixed to refrigerator shelves (includes mounting brackets, nuts and bolts)
- Storage boxes function as a locked refrigerator drawer – once affixed to refrigerator shelves, boxes slide out from unlocked lid
- Tough, durable polyethylene construction
- Available in ● grey, ● light blue, or ● red – please specify colour when ordering


Item #	Description	Inside Dimensions (W x H x D)
3712	E Single Lock	6-1/2 x 2 x 9-1/8"
3713	Single Lock	6-1/2 x 3 x 9-1/8"
3714	F Single Lock	6-1/2 x 4-1/2 x 9-1/8"



Hinged Locking Refrigerator Boxes

- Ideal for use in facilities without access to full-size or locking refrigerators
- Self-locking nuts and bolts cannot be removed unless unit is open and unlocked – ensures safe and secure control over substances
- Available with single, double, or combination keyless locks
- Mounting accessories included

Item #	Size	Dimensions (L x W x H)	Lock
3746-01	A Compact	8-3/8 x 2-7/8 x 4-1/2"	Single Lock
3747-01	Compact	8-3/8 x 2-7/8 x 4-1/2"	Double Lock
3748-01	Small	12 x 4-1/2 x 6"	Single Lock
3749-01	B Small	12 x 4-1/2 x 6"	Double Lock
17843	C Small	12 x 4-1/2 x 6"	Keyless Entry Digital Lock

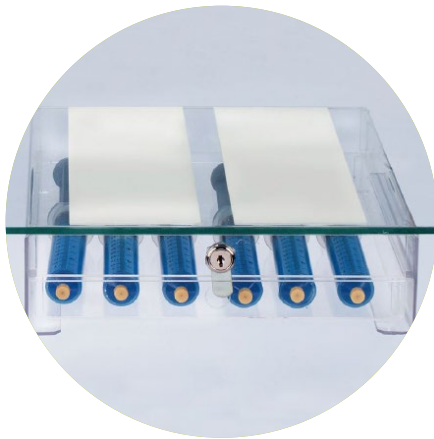

Two-Sided Tape for all Refrigerator Boxes

- Flexibility - Mount refrigerator boxes virtually anywhere. Double-sided tape works great on carts, countertops, in narcotic safes or anywhere a small box may be needed.

Item #	Dimensions (W x L)
9934	4 x 11"

Refrigerate Zippit Bag

- Preprinted Message - Alert users to temperature sensitive contents.
- Thickness - 2.5-mil
- Closure - Reclosable zipper

Item #	Size	Qty
7629-10	5 x 8"	100
18239	3 x 5"	100
19547	12 x 15"	100



MonitorMark™ Product Exposure Indicators

- Ideal for use when transporting temperature-sensitive items
- Tags monitor temperature by changing colour to visually depict if acceptable temperature limit has been exceeded
- Records time exposed to unacceptable temperatures
- If an item can tolerate limited exposure to elevated temperatures, the results on the tags will help determine whether to accept or reject the item
- When the maximum temperature of the tag is exceeded, a blue colour appears in the window furthest to the left
- Colour change is irreversible
- Attaches with an adhesive back to most clean, dry surfaces
- Measures : 3 7/8 x 3/4"
- 10 per package

Item #	Temperature Range	Exposure Time
8201-01	5°C (41°F) and above	48 Hours
8205-01	10°C (50°F) and above	48 Hours
8207-01	10°C (50°F) and above	1 Week
8208-01	31°C (88°F) and above	1 Week



Insulated Boxes

- Disposable, lightweight cooler to transport and ship temperature-sensitive items
- Leak-resistant gusseted liner made from FDA-approved materials
- Liner's adhesive lip tape seals the contents for maximum protection
- Boxes are shipped flat; each has instructions for assembly

Item #	Dimensions (W x H x D)	Qty
17878	8 x 8 x 8"	5

Insulated Transport Boxes


- Cost Effective - Keep medications at a stable temperature during transport.
- Protects Drug Integrity - 1 1/2" thick insulated walls are ideal for transporting 24 to 48-hour shipments.

Item #	Dimensions (W x H x D)	Qty
19808	11 x 10 x 9	1
19807	15-1/2 x 12 x 15-1/2	1



AkroBins

- Durable and long-lasting AkroBins are the industry standard for plastic storage bins
- Heavy-duty, sturdy, one-piece construction is water, rust and corrosion-proof
- Many great organization options are available - stack bins, hang on racks, wall panels, louvered panels, or use on mobile carts
- Reinforced ribs prevent bins from spreading or losing shape
- Large front label slot allows for labelling and relabelling system.
- Take advantage of colour options available to create a colour-coded organization system to control inventory
- Ideal for Kanban visual organization systems and 5S Lean initiatives
- Customize organization system to meet your unique needs – select from standard size bins, super-size bins, bin dividers, clear or amber lids, and many hanging organization strategies

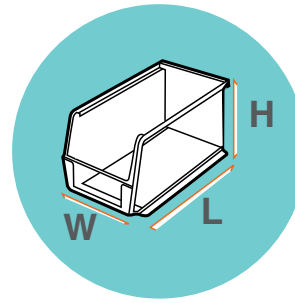

Refer to this outside measurement diagram for all AkroBins® and Insight bins by Akro-Mils®

Item #	Outside Dimensions			Capacity (lbs.)	Qty
	Length	Width	Height		
1400	5-3/8"	4-1/8"	3"	10	24
1401	10-7/8"	5-1/8"	4"	30	12
1403	14-3/4"	5-1/2"	5"	60	12
1405	7-3/8"	4-1/8"	3"	10	24
1410	10-7/8"	5-1/2"	5"	30	12
1412	10-7/8"	11"	5"	60	6
1413	10-3/4"	8-1/4"	7"	60	6
1415	14-3/4"	8-1/4"	7"	60	12
1417	10-7/8"	16-1/2"	5"	60	6
1420	14-3/4"	16-1/2"	7"	75	6
1434	18"	8-1/4"	9"	75	6
1438	18"	16-1/2"	11"	75	3



Amber lids are ideal for storing light-sensitive medication or IV's

Super Tough Bins with Clear or Amber Lids

- Select from clear or amber lids, attached or sold separately
- Shield Supplies: Transparent lids keep contents dust-free and protected.
- Easy Content ID: Label slots hold fanfold, roll or laser labels
- Compatible - Waterproof, oil and chemical resistant. Lids keep contents protected
- Bins with lids can still be stacked (however lids cannot be opened when stacked)
- Specify colour when ordering

Item #	Description	Lid Colour	Fits Akro Bin
1672	Bin with Lid	Clear	#1400
1673	Bin with Lid	Clear	#1405
1676	Bin with Lid	Clear	#1410
1558-01	Bin with Lid	Amber	#1410
1555	Bin with Lid	Clear	#1420
1556	Bin with Lid	Amber	#1420

Dividers for AkroBins

- Divides bin front to back
- Material: Made of industrial grade polymers
- 6 dividers per package

Item #	Fits Bin Number(s)	Colour
1931	1400	Black
1933	1405	Black
1935	1410, 1412	Black
1936	1413	Black
1937	1415, 1420	Black
1968	1401	Clear
1973-01	1403	Clear



Super-sized AkroBins

- Large size conducive for large, heavy items
- Sturdy - Reinforced sides keep bins from spreading.
- Weight Capacity - 60 - 75 lbs.
- Available in ● red, ● yellow or ● blue - please specify colour when ordering
- **Note:** Bins can't be hung from panels or rails.



Item #		Outside Dimensions			Capacity (lbs.)	Qty
		Length	Width	Height		
1404	A	20"	12-3/8"	6"	60	4
1406		20"	12-3/8"	8"	60	3
1407	B	20"	12-3/8"	12"	75	2
1408	C	20"	18-3/8"	12"	75	1
1409		29 1/4"	18-3/8"	12"	75	1



Shelf Bins

- Fits standard 12", 18" and 24" shelving (see length below)
- Customize to fit organization needs using bin dividers or bin cups (sold separately)
- Durable polypropylene construction allows Shelf Bins to be fully loaded without bulging or cracking
- Waterproof one-piece construction is unaffected by grease or oil
- Molded label holder for identifying contents
- **Note:** Bins are not made for stacking

Item #		Outside Dimensions (L x W x H)	Case Qty	Small/ Large bin cups fit per bin	Divider Item #
1435	A	11-5/8 x 2-3/4 x 4"	24	3/-	-
1440		11-5/8 x 4-1/8 x 4"	24	5/2	1941 (7)
1445	B	11-5/8 x 6-5/8 x 4"	12	9/2	1943 (7)
1447		11-5/8 x 8-3/8 x 4"	12	11/4	1947 (7)
1448		11-5/8 x 11-1/8 x 4"	12	15/6	1948 (7)
1450		17-7/8 x 4-1/8 x 4"	12	8/3	1941 (7)
1455		17-7/8 x 6-5/8 x 4"	12	15/6	1943 (7)
1457		16-1/2 x 7-1/8 x 4"	12	15/6	1947 (7)
1456		16-1/2 x 11-1/8 x 4"	12	25/12	1948 (7)
1451	C	22-1/8 x 4-1/8 x 4"	12	11/4	1941 (7)
1458		23-5/8 x 6-5/8 x 4"	6	21/8	1943 (7)
1459		23-5/8 x 8-3/8 x 4"	6	25/8	1947 (7)
1454		23-5/8 x 11-1/8 x 4"	6	35/16	1948 (7)

Shelf Bin Dividers


- Polypropylene dividers create compartments within Shelf Bins
- Each bin can hold up to 7 dividers to create 8 compartments (except for item #1435, not compatible with dividers)
- Six dividers per pack

Item #	Colour	Width	Fits Shelf Bin #
1941	D Black	4"	1440, 1450 and 1451
1942-01	Clear	4"	1440, 1450 and 1451
1943	Black	6"	1445, 1455 and 1458
1947	Black	8"	1447, 1457 and 1459
1948	Black	11"	1448, 1456 and 1454

Bin Cups (for Shelf Bins)

- Use to keep small parts separated and organized within Shelf Bins
- White, made of industrial grade polymers

Item #		Outside Dimensions (W x H x D)	Qty
1949	E	3-1/4 x 3 x 2"	48
1950		5 x 3 x 2-3/4"	24

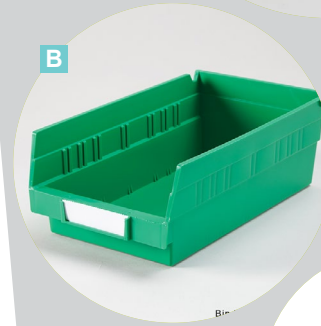

Mesh Basket

- Compact - Plastic baskets in three sizes provide multiple storage options for organizing vials and other pharmacy supplies. Baskets nest when not in use.
- Smart Design - Mesh construction allows airflow and easy content identification.



Item #	Dimensions (W x H x D)
19921W A	6-11/16 x 2-3/4 x 9-13/16"



Tilters® Tilt Front Bins

- Modular bins – stack and customize to fit your storage and organization requirements
- Front opens at a 45 degree angle for easy access
- Clear fronts provide easy visual access to inventory
- Moulded in tough ○ beige or ○ white plastic (specify colour when ordering)

Item #	Overall Dimensions (W x H x D)	Compartment Dimensions (W x H x D)	# of Compartments per unit
1203A B	235/8 x 41/2 x 35/8"	3-1/8 x 2-3/4 x 2-5/8"	6
1205E C	235/8 x 61/2 x 51/4"	4 x 4-1/2 x 3-3/4"	5
1207E D	235/8 x 81/8 x 65/8"	5 x 5-1/4 x 5"	4
1210A E	235/8 x 91/2 x 73/4"	7-3/4 x 23-5/8 x 9-1/2"	3
1215 F	235/8 x 137/8 x 113/4"	11-7/8 x 23-5/8 x 13-7/8"	2
1213 G	113/16 x 137/8 x 113/4"	10-1/2 x 13-1/4 x 8-1/4"	1

Stainless Steel Trays

- Heavy-gauge stainless steel tray can be used as a processing tray, medical supply organizing tray or a pass-through tray to the cleanroom
- Ideal for the pharmacy and lab use
- Seamless, non-corrosive metal tray has open corners and easy-grip handles and nests when empty
- View all stainless steel trays on page 32

Item #	Size	Dimensions (W x H x D)
18779	Small	11-11/16 x 2-1/16 x 6-3/8"
18665	Large	15 x 2-1/4 x 10-3/16"
18783	Shallow	12 x 1/2 x 9"



Rx Fill Tray

- Rx fill trays help keep individual prescription components together from arrival at the drug store to the prescription pick up counter
- Trays have easy-grip handles, nest when empty to save space
- Available in ● blue, ○ clear, ● green, ● red, ○ white and ● yellow - specify colour when ordering

Item #	Outside Dimensions (W x L x H)
17487	6-7/8 x 10-1/2 x 2-1/2"

Prescription Boxes

- Storage - Use for storing archives or keep it on the counter for organizing incoming prescriptions
- Shipping - The paperboard box is shipped flat and is easy to assemble.

Item #	Size	L x W x H	Qty	UPC
11194	A	Small 13 x 6 1/4 x 4 1/2"	1	063636711948
10196	B	Large 16 1/8 x 5 3/4 x 7 1/2"	1	063636701963



Prescription File Folders

- Keeps prescription slips organized for quick reference and storage
- Double fold file folders hold approximately 100 prescription sheets
- When folded, the spine is 1/2" deep and can be used to label or identify contents
- Measures: 5 1/2 x 13"
- Accommodates: 5 1/2 x 4" prescription slips

Item #	Qty
1976	C 100
19464	D 100

Hanging Prescription Bags

- Simplify prescription organization and retrieval
- Made of tough, clear 4 mil plastic for extreme durability
- Fits on any rod with a diameter of 1 cm or smaller

Item #	Size (W x H)	Qty
17540 A	10-1/2 x 7-1/2"	10
17541	12 x 9"	10
17542	14 x 12-1/2"	10

Wall Rack for Prescription Bags

- Simple solution to prescription organization
- Saves time, improves accuracy and enhances customer satisfaction
- Aluminum frame is 3 ft. wide and holds five 1 cm thick rods
- Each rod holds approximately 8-10 bags per foot or as many as 150 bags per rack
- Easy assembly, all hardware included
- Measures: 36" W x 57½" H x 7½" D
- Use with hanging bags and ID tags (sold separately)

Item #	Description	Qty
17543	Wall Rack for Prescription Bags	1 frame, 5 rods
17544 B	Green ID Tags	25

Bag Caddy for Hanging Prescription Bags

- Bag caddy designed for use with hanging prescription bags
- Holds up to 40 hanging prescription bags
- Ideal for use in pharmacies, to organize bags to be reused
- Measures: 6"W x 16.5"H x 7"D

Item #
17545 C



Stock Prescription Receipt Pads

- Three popular sizes and designs
- Padded in 100's, sold in 1000's

Item #	Description	Qty
PRP-3 A	4-1/4 x 7"	10 pads of 100 sheets
PRP-6 B	3-1/4 x 7"	10 pads of 100 sheets
PP-8 C	4-1/4 x 6" (Hole Punched)	10 pads of 100 sheets





Rx Tape

- 3 sizes are individually packaged for convenient storage
- Two core sizes to fit different tape dispensers

Item #	Width	Length	Core Size
T1-1	1"	33 yards	1"
T1-3	1"	72 yards	3"
T1-5	1-1/2"	72 yards	3"
T1-7	2"	55 yards	3"

Tape Dispensers

Single Roll Tape Dispenser

- Accepts tape up to 1" wide
- Core adapter accommodates 1" and 3" cores


Double Roll Tape Dispenser

- Accepts tape up to 2" wide
- Core adapter accommodates 3" cores

Triple Roll Tape Dispenser

- Accepts tape up to 3" wide
- Can run 3 rolls of 1" wide tape or two rolls of 1 1/2" tape individually, 3" core

Item #	Description
TD1	D Single Roll Tape Dispenser
GF2306	E Double Roll Tape Dispenser
GF2336	F Triple Roll Tape Dispenser

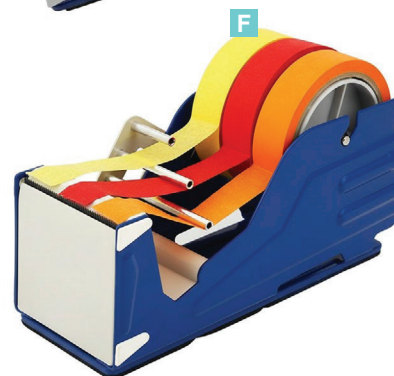

Pharmacy Prescription Labels

- Preprinted in yellow & pink
- Smudge-free topcoat for clarity and durability
- Includes space to print up to 5 auxiliary labels
- Fully compatible with Kroll, Fillware, PharmaClik, and PropelRx systems - contact us to learn more
- Fully compatible with desktop printers such as Zebra ZD and GK. Contact us for more information

Item #	Dimensions (W x H x D)	Quantity
80411	A 2" x 4.5"	500 labels per roll
80412	B 3.375" x 4.25"	900 labels per roll

Storage Information Labels



1-15G



1-324



P-5



P-12



P-13



2281



2280



AHSPT012



AHSPT026



CL-41



AHSPT029



CL-241




LPH-239



2960



AHSPT028

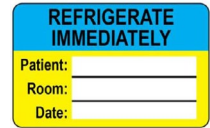


CL-713

Pharmacy Consultation Labels



LPH-50

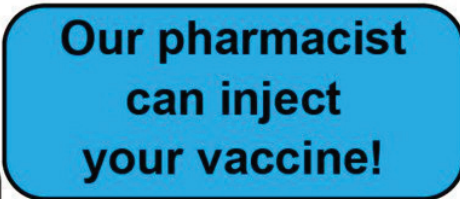

CL-1146




CL-1369



CL-1532



CL-696

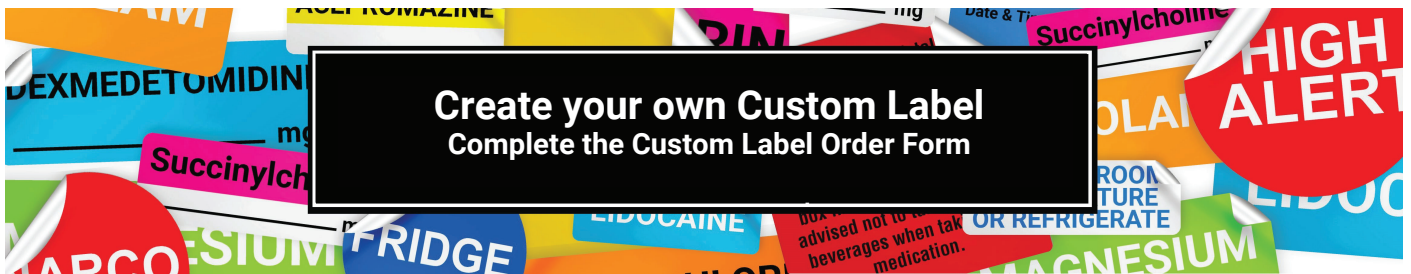

Quick Order Form



Vaccine Essentials



Please Email or Fax Completed Form to: orders@pharmasystems.com | Fax: 1.888.475.7155

Part Code	Description	Pack Size	Enter Order Quantity
Safety Needles			
<u>V0205*</u>	25G x 1" Adult Safety Needles	100	
<u>V0206*</u>	25G x 1-1/2" Higher BMI Population Safety Needles	100	
<u>V0207*</u>	25G x 5/8" Adolescents/Infant Safety Needles	100	
Luer Lock Syringes			
<u>V0201*</u>	1ml Luer Lock Syringe LDS (low dead space), without Needle	100	
<u>V0203</u>	3ml Luer Lock Syringe, without Needle	100	
Sharps Disposal			
<u>9604-01</u>	Sharps Container, 1 Quart	1	
<u>9601-01</u>	Sharps Container, 5 Quart (horizontal drop)	1	
<u>9601-06</u>	Sharps Container, 5 Quart (horizontal drop)	20	
Gloves			
<u>GNSMS</u>	Nitrile Gloves (Blue), Small	100	
<u>GNSMM</u>	Nitrile Gloves (Blue), Medium	100	
<u>GNSML</u>	Nitrile Gloves (Blue), Large	100	

*Low Dead-Space. Learn more here: pharmasystems.com/vaccine-clinic-supplies

PHARMACY NAME:		CONTACT NAME:	
ADDRESS:		PHONE:	
CITY:		FAX:	
PROVINCE:	POSTAL CODE:	EMAIL:	

PHARMASYSTEMS CUSTOM LABELS

SELECT LABEL SIZE & QUANTITY

LABEL ITEM NO.

QTY:

BOX(ES)

SELECT BACKGROUND COLOUR

- BLACK**
- WHITE**
- RED**
- NAVY BLUE**
- SKY BLUE**
- YELLOW**
- GREEN**
- ORANGE**
- PURPLE**

- FLUORESCENT RED**
- FLUORESCENT ORANGE**
- FLUORESCENT YELLOW**
- FLUORESCENT GREEN**
- FLUORESCENT PINK**

Please note: Only black text is available with fluorescent colours

SELECT TEXT COLOUR

- BLACK**
- WHITE**
- RED**
- NAVY BLUE**
- SKY BLUE**
- YELLOW**
- GREEN**
- ORANGE**
- PURPLE**

INPUT LABEL INFORMATION (PLEASE PRINT CLEARLY)

ADD A SYMBOL (OPTIONAL)

TO INCLUDE A COMPANY LOGO OR CUSTOM SYMBOL, PLEASE SUPPLY A VECTOR FILE (AI, EPS, PDF, SVG).



SYMBOL NO.:

PHARMACY NAME:		CONTACT NAME:	
ADDRESS:		PHONE:	
CITY:		FAX:	
PROVINCE:	POSTAL CODE:	EMAIL:	
DATE:	WHOLESALER:	ACCOUNT NO.:	



Please email or fax the completed order form to us. Kindly only return this page. Please note: we'll send a label proof to the email provided for approval. Labels will be ordered only after approval is confirmed.

PharmaSystems Inc.

pharmasystems.com
orders@pharmasystems.com
fax: 1.888.475.7155

